


Handpiece Care and Lubrication

CARE

- ◆ Clear chip air and water tubes with clean out wire.
- ◆ Clean hose connector threads with gauze and brush.
- ◆ Never exceed 38psi air pressure during operation.
- ◆ On standard chuck models, avoid over tightening or over loosening chuck.
- ◆ Do not exceed 275 degrees Fahrenheit for 20 minutes in autoclave.
- ◆ Never operate without bur secured in chuck.

LUBRICATION

- ◆ Insert bur into chuck and tighten.
- ◆ Inject lubricant into drive air tube—spray or drops.

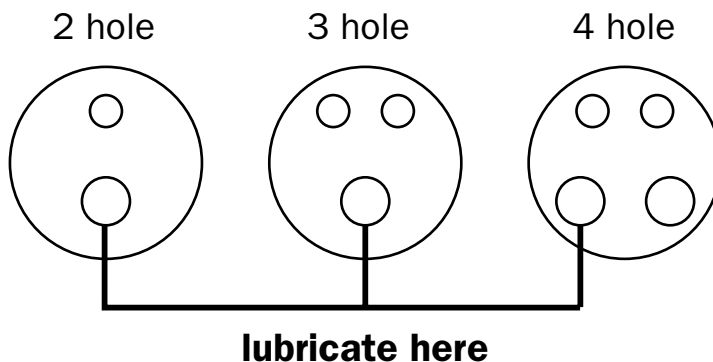

- ◆ Connect air supply line.
- ◆ Expel any excess lubricant into clean paper towel.
- ◆ Repeat above process if discharge is discolored.
- ◆ Sterilize handpiece.